

Surgimap[®]

The Physician Driven Imaging Solution[®]

A Nemaris, Inc. Product

OsiriX Connection

For Surgimap version 2.2.13 and higher

DATE: 11-3-2017

VERSION 1.6

475 Park Ave S, 11th Fl, New York, NY 10016

Connecting with OsiriX


1. OPEN AND LOG INTO SURGIMAP.



2. CLICK SETTINGS  IN THE TOP RIGHT CORNER, THEN ON 'DICOM NODES'.

3. SELECT THE 'SETUP LOCAL OSIRIX' BUTTON TO CONFIGURE SETUP WITH 1 CLICK.


NOTE: OSIRIX MUST BE CLOSED DURING THIS SETUP PROCESS.

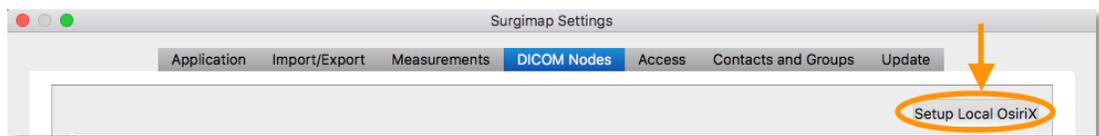


4. OPEN OSIRIX. NOTE: OSIRIX MUST BE OPEN DURING THE IMPORT INTO SURGIMAP.




5. SELECT THE 'OSIRIX LOCALHOST' LOCATION. SEARCH FOR A PATIENT BY NAME, PATIENT ID, MODALITY, ETC. PRESS 'SEARCH'.

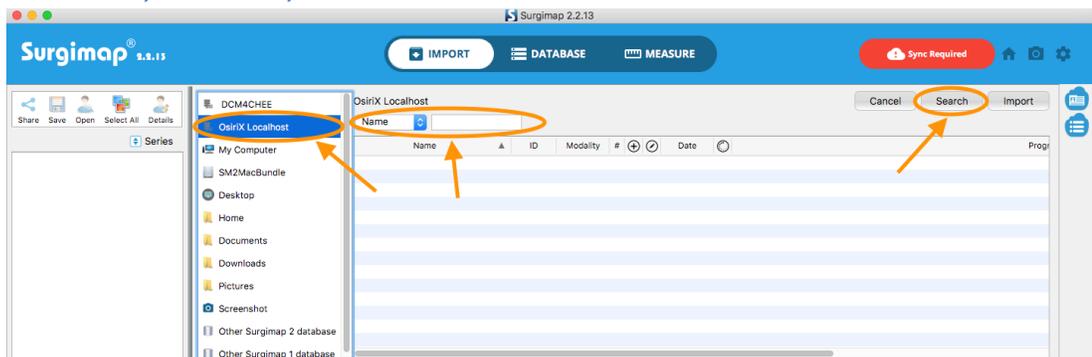


6. CLICK ON THE PATIENT, THEN ON 'IMPORT'.

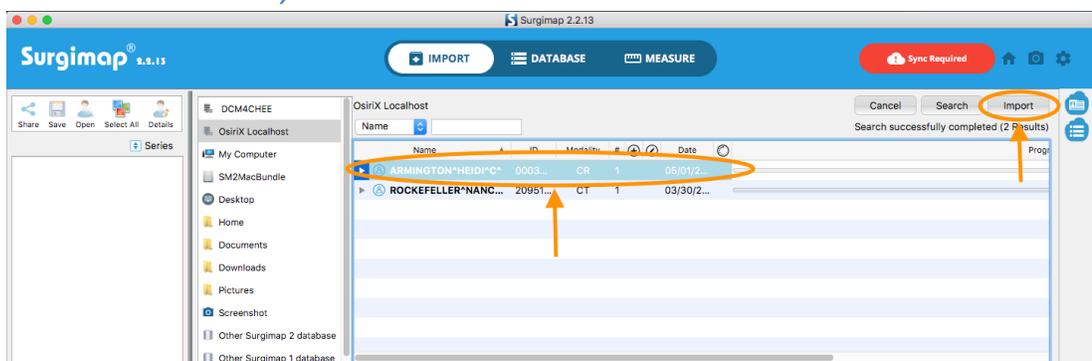

7. *IMPORT IS COMPLETED ONCE THE PROGRESS BAR REACHES 100%.*

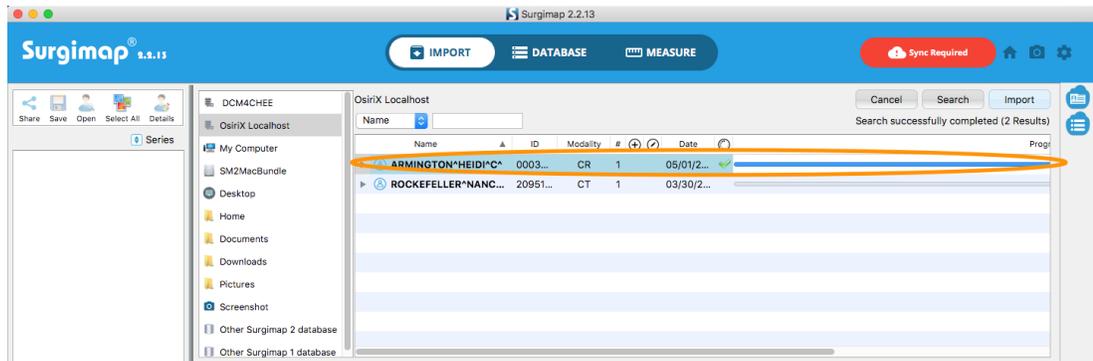

8. *GO TO YOUR DATABASE TO LOCATE THE IMPORTED PATIENT AND IMAGES.*

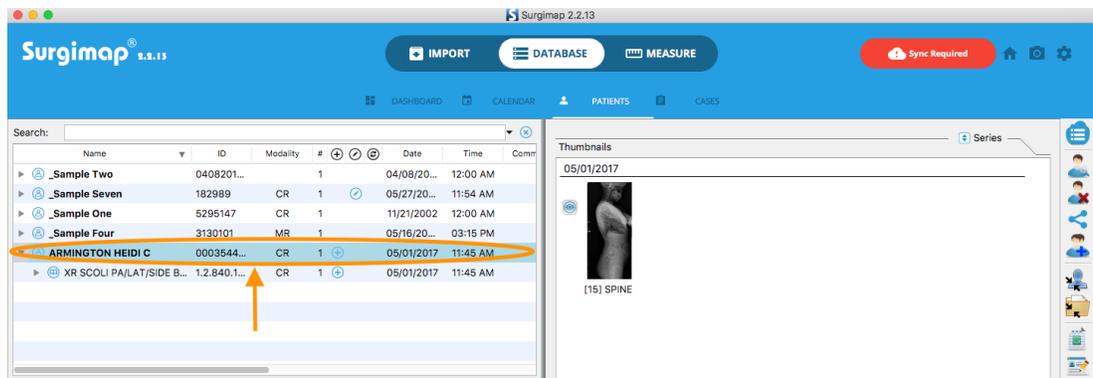

IF YOU ARE STILL EXPERIENCING TROUBLE, PLEASE CONTACT US AT SUPPORT@SURGIMAP.COM